

Product Catalog I

2009 - 2010



BIOSEPAR
Gesellschaft für Medizin- und Labortechnik mbH
Head office
Am Industriepark 3, D-84453 Mühldorf am Inn
Export office Munich
Otto-Heilmann-Str. 2, P.O.Box 1344, D-82027 Grünwald/Munich
Phone: +49 (0) 8631 167 48 55, Fax: +49 (0) 8631 167 48 57
E-mail: mail@biosepar.de
Internet: www.biosepar.de

Important informations about the Biosepar ParasiTrap[®] diagnostic system

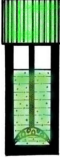

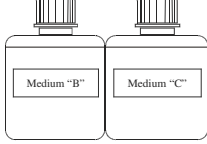
The **Biosepar ParasiTrap[®] diagnostic system** is the result of the long research work at the Max von Pettenkofer Institute for Hygiene and Medical Microbiology, University of Munich. This system was developed to ensure better and safer results in the diagnostic and to raise the safety in working with the infectious stool samples in an effective and permanent way. The rational working procedure opens the possibility to work with great sample series in the shortest possible time, saving processing time and costs.

The **Biosepar ParasiTrap[®] diagnostic system** is worldwide market leader concerning the reproducibility of the results; standardibility and sensitivity in detect parasites, being used with great success in many institutes and laboratories in Europe and overseas. It complains with all necessary requirements from the modern quality management.

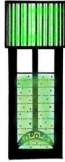

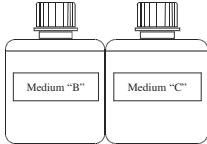



Not only has the high accuracy in the diagnosis but also the excellent quality of the sight field in the microscope helped to obtain exceptional results in the quickest way. The handling is very simple and the diagnosis procedure should be interrupted even for a longer time at any step. No refrigeration of the samples is necessary. Using the system as recommended in the manual eliminate any danger of infections and perturbing emissions (ill smells or organic solvents), the maximal work place concentration value stay at normal level. The many advantages from the **Biosepar ParasiTrap[®] diagnostic systems** will reveal itself during the use. For any special question related to this system, we will be glad to help and support you. We also provide personal trainings in your laboratory to ensure that your personal can enjoy the full capacities and advantages that the **Biosepar ParasiTrap[®] diagnostic system** offers. All participants will receive the Biosepar Certificate attesting this training.

PARASITOLOGICAL DIAGNOSTICS FOR STOOL

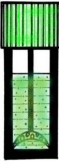

ParasiTrap[®] Super Combipack Comfort System FixSepar[®] ECO

Order-No.: 8484 ECO		(unit = 50 pieces/box)	Super Combi System	
	+		+	
Processing tube I		Processing tube II		Medium B, Medium C
(filled with 3,5 ml Medium FixSepar ECO)		(with integrated filter system)		(incl. the sufficient quantity of media)




ParasiTrap[®] Premium Combipack Comfort System FixSepar[®] ECO

Order-No.: 8486 ECO		(unit = 50 pieces/box)	Premium Combi System	
	+		+	
Processing tube I		Processing tube II		Medium B, Medium C
(filled with 3,5 ml Medium FixSepar ECO)		(with integrated filter system)		(incl. the sufficient quantity of media)
		+		
				
Sample spoon		One-way pipette		Cotton wool stick

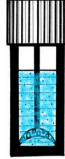

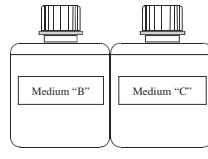
ParasiTrap® FixSepar® ECO System for Fecal Diagnostics

Order-No.: 8488 ECO	(unit = 50 pieces/box)	Filled transport / processing and filter system
	+	
Processing tube I (filled with 3,5 ml Medium FixSepar ECO)		Processing tube II (with integrated filter system)

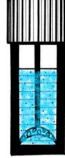

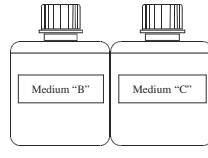
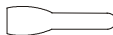


ParasiTrap® Single Tubes and Vials FixSepar® ECO

Order-No.: 8490 ECO	(unit = 50 pieces/box)	Processing tube I
Processing tube I (filled with 3,5 ml Medium FixSepar ECO)		
Order-No.: 8491	(unit = 50 pieces/box)	Processing tube II
Processing tube II (with integrated filter system) Filter colours: black or green	 	

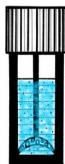

ParasiTrap[®] - Super Combipack Comfort - System

Order-No.: 8484 AF/SAF		(unit = 50 pieces/box)	Super Combi System	
	+		+	
Processing tube I (filled with 3,5 ml Medium AF/SAF)		Processing tube II (with integrated filter system)		Medium B, Medium C (incl. the sufficient quantity of media)


ParasiTrap[®] - Premium Combipack Comfort - System

Order-No.: 8486 AF/SAF		(unit = 50 pieces/box)	Premium Combi System	
	+		+	
Processing tube I (filled with 3,5 ml Medium AF/SAF)		Processing tube II (with integrated filter system)		Medium B, Medium C (incl. the sufficient quantity of media)
		+		
				
Sample spoon		One-way pipette		Cotton wool stick

ParasiTrap® Fecal Diagnostic System


Order-No.: 8488 AF/SAF	(unit = 50 pieces/box)	Complete system
	+	
Processing tube I (filled with 3,5 ml Medium AF/SAF)		Processing tube II (with integrated filter system)

ParasiTrap® Single Tube

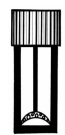

Order-No.: 8490 AF/SAF	(unit = 50 pieces/box)	Processing tube I
Processing tube I (filled with 3,5 ml Medium AF/SAF)		

Special production as required
ParasiTrap[®] Empty System

Order-No.: 8489	(unit = 50 pieces/box)	Processing tube I
------------------------	------------------------	--------------------------


Processing tube I (unfilled)	
--	--

Order-No.: 8487	(unit = 50 pieces/box)	Empty System
------------------------	------------------------	---------------------


	+	
Processing tube I (unfilled)		Processing tube II (with integrated filter system)

Biosepar Media for **ParasiTrap**[®] Diagnostic System


Order-No.: 8487/A		Medium A
Transport, processing and fixative solution		

	8487/A F	Fixative solution with formaldehyde
	8487/A SAF	SAF solution with formaldehyde
	8487/A B	Bailenger solution

Order-No.: 8487/B		Medium B
Separation solution		

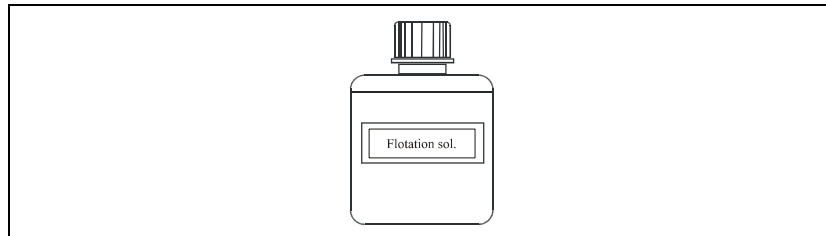
	Ethyl Acetate
--	----------------------

Order-No.: 8487/C		Medium C
Staining solution		

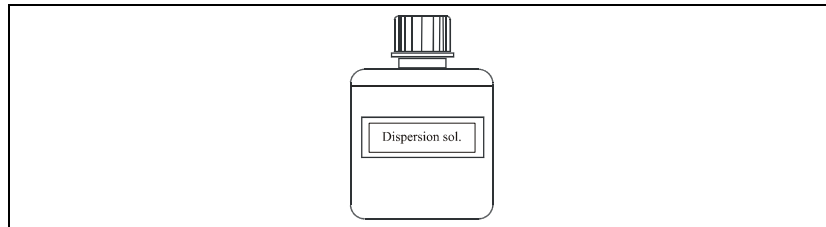
	ParasiTrap Lugol (special modified and made for parasitological purposes)
---	--

Special work and fixation solutions for **ParasiTrap[®]** diagnostic system

Order-No.: 8500	Flotation solution for the use in the flotation method
------------------------	--



Order-No.: 8535	Dispersion solution for sample dilution
------------------------	---



Order-No.: 8536	Isopropanol 70% desinfection solution
------------------------	---

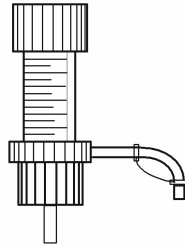


Order-No.: 8514	Hemolyse Reagent
------------------------	-------------------------

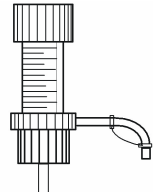


Dispenser

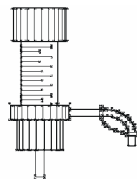
Order-No.: 8495/A	(unit: 1 piece)	Dispenser
--------------------------	-----------------	------------------

2,0-10,0 ml (for Medium AF, SAF, Bailenger)	
---	--


Order-No.: 8495/B	(unit: 1 piece)	Dispenser
--------------------------	-----------------	------------------


1,0-5,0 ml (for Medium B)	
------------------------------	---


Order-No.: 8495/C	(unit: 1 piece)	Dispenser
--------------------------	-----------------	------------------

0,1-1,0 ml (for Medium C)	
------------------------------	---

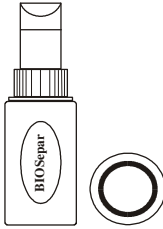
Accessories for working with stool and collecting stool samples (Parasitology)

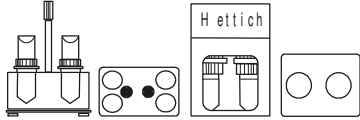
Order-No.: 8492	(unit = 50 pieces)	Sample spoon
to take stool samples in difficult situations		

Order-No.: 8493	(unit = 50 pieces)	One-way pipette
to take liquid stool samples and for the processing of the ParasiTrap method		

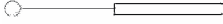
Order-No.: 8494	(unit = 50 pieces)	Cotton wool stick
for the processing of the sample in the ParasiTrap method		

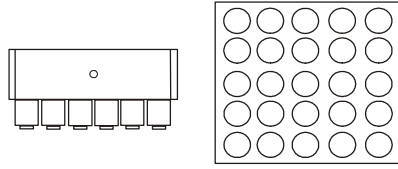
Accessories for centrifugation

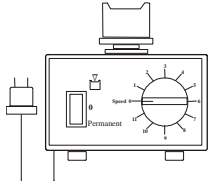
Order-No.: 8504	(unit = 2 piece)	Adapter for tubes
Biosepar tubes adapter for centrifuge insert		

Order-No.: 8510	(unit = 2 piece)	Centrifuge insert
for the stool diagnostic system (Please indicate in your order the mark, serial number and model of your centrifuge!)		

Accessories for the flotation method

Order-No.: 8501	(unit = 1 piece)	Wire loop
for the flotation method		


Order-No.: 8502	(unit = 1 piece)	Tube support
for the use in the parasitological diagnostic, place for 25 tubes (Can be delivered in different colours, autoclavable)		

Order-No.: 8503	(unit = 1 piece)	Shaker
for the Biosepar method in the medical or research laboratory		



Accessories for microscopy


Order-No.: 8506		Sample holder
------------------------	--	----------------------

Cutted edge, Opaque field beside: 76 x 26 mm, 0,8-1,0 mm thickness	
---	--

Order-No.: 8507		Covering glas
------------------------	--	----------------------

24 x 36 mm Special recommended for the parasitology	
---	--

Order-No.: 8509		Covering glas
------------------------	--	----------------------

24 x 60 mm Special recommended for the parasitology	
---	---

Screw caps for Biosepar tubes

Order-No.: 8482	Standard screw cap for Processing tube I and II
------------------------	--

Screw cup for the ParasiTrap system	
--	--

General conditions of sale

1. General remarks

Our deliveries, services and offers shall be affected solely on the basis of these terms and conditions. Deviating terms and conditions of the customer shall not apply without our express written consent, even if we do not expressly oppose them. The sales conditions shall also apply to all future business relations. Any deviation expressly agreed to from individual provisions of these sales conditions shall not affect the validity of the remaining provisions.

2. Offer - quotations

Our quotations are made subject to change without notice. Unless otherwise agreed, our order confirmation shall be decisive for the scope of our contractual performance. The specifications, drawings, illustrations, images, technical data, measurements and performance descriptions in our brochures, description and quotations shall be deemed approximate, unless they are expressly specified as binding. We expressly reserve the right to make any changes to the specification of our products.

We reserve property rights to tools and installations. The same shall apply to documents specified as "confidential". They may only be disclosed to third parties with our express prior consent.

3. Prices - terms of payment

Unless otherwise specified, prices are quoted "ex works" plus the legal percentage of Value Added Tax valid on the invoice date.

Payment shall be effected subject to the specified terms of payment. If no terms have been specified, the purchase price shall be payable within 14 days from the invoice date without deductions. If bills of exchange or cheques are accepted in payment due to a prior agreement, any discount and collection expenses, etc., shall be charged to the customer. Bills of exchange and cheques shall only be accepted unaccounted performance.

If the customer fails to pay within the payment deadline we shall be entitled to charge interest equalling the rate of usual bank debit interest but at least 8% over the basic interest rate of the invoiced amount. In the event that the production costs of goods or services that are to be delivered or rendered later than 4 months after conclusion of the contract go up by at least 5% due to an increase in raw material prices, we reserve the right to modify the originally agreed prices correspondingly.

4. Terms of delivery - default on delivery

The quoted delivery terms are not fixed deadlines, unless expressly agreed otherwise. We shall be entitled to make part deliveries. All deliveries shall be ex works and travel at the customer's risk even if carriage-free delivery has been agreed.

In the event of strikes, lock-outs, scarcity of power or resources, transport and unavoidable operational disruptions as well as other cases of force majeure, we shall be entitled, at our discretion, to either extend the term of delivery appropriately or to withdraw from the contract in whole or in part.

In the case of a default on delivery, for which we are at fault the buyer may withdraw from the contract after expiry of a reasonable grace period to be set by him in writing. Art. 6 shall apply to claims for damages.

5. Warranty

The customer shall give us written notice of any defects immediately but no later than within 14 days of receipt of delivery. Written notice of defects that cannot be detected despite careful inspection within this period shall be given to us in writing immediately after they have been detected.

If the delivered merchandise is defective at the time of passing of the risk, we shall either deliver a replacement or remedy the product at our discretion. Multiple remedies shall be admissible.

If belated performance/remedy fails, the customer - unless otherwise specified by the warranty - shall be entitled at his discretion to either withdraw from the contract or demand a reduction of the purchase price. In the case of a minor violation of the contract, in a particular in the case of minor defects, however, the customer shall have no right of withdrawal. Art. 6 shall be applying to claims for compensation of damages and expenses due to defects.

The warranty period for our products shall be 12 months, starting on the date of delivery.

6. Limited liability

For wilful intent and gross negligence on our part and in the case of culpable damages to life, body or health we shall accept liability according to legal regulations. In the case of a violation of important rights and obligations, essential or fundamental contractual or fundamental incidental or protection duties we shall accept liability in the case of simple negligence, limited to typical, immediate and predictable average damages. Any further liability, notwithstanding the legal nature of the raised claim, shall be ruled out. This shall apply in particular to claims for damages in the case of culpability on conclusion of the contract, due to other violations of duties or to tortious claims for compensation of material damages according to § 823 of the German Civil Code (BGB). Claims of the customer arising from product liability and assumed warranties shall not be affected by limited liability. We shall also be under no liability in the case of violations of duties by our legal representatives and vicarious agents.

7. Reservation of title

We shall retain title to the delivered merchandise until all claims - even future claims - resulting from our business relations have been settled. To secure our rights, any claims arising to the customer from re-sale of the reserved merchandise to third parties shall already be transferred to us in the amounts charged to the third parties. The direct debit mandate issued to the customer shall expire if the fails to meet his payment obligations in due and proper form. On our request, the customer shall make known to us the parties owing the transferred claims and inform them of the assignment. If the securities exceed our claims to be secured by more than 10%, we shall release any excess securities on demand.

8. Conclusions of contracts

Any agreement with our agents shall require our express written confirmation. This shall apply in particular to orders accepted by the agents.

9. Governing law - official language - jurisdiction

These general conditions of sale and all legal relations between seller and buyer shall be governed solely by the law of the Federal Republic of Germany. The Uniform Law on the International Sale of Goods shall not apply.

If these general conditions of sale are disclosed to the customer in another language apart from German, only the German text shall be decisive in the case of translation / interpretation differences. Any translation into another language shall solely serve the purpose of better understanding.

The jurisdiction, also for actions on bills or exchange and cheques, for possible disputes with fully qualified merchants, legal persons under public law or public assets shall be - depending on factual jurisdiction - the Local Court of Munich, Bavaria, Germany, or at our discretion, the customer's general legal venue.